Operator Manual

JOEY™

VSL-4400

02/04/2022

P/N 4400-O

English/ français

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The JOEY™ barrier is protected by Patent Nos. 6,726,435 and 8,403,625.

www.bruno.com

Thank you for purchasing a VSL-4400.

The information contained in this manual is provided with your safety in mind. Please read and follow the instructions to ensure safe and trouble-free operation. Throughout this manual, **safety precautions** are provided to identify potentially hazardous situations and to advise how to avoid them.

WARNING: Indicates a potentially hazardous situation that, if not avoided, could cause serious bodily injury and/or property damage.

CAUTION: Indicates a potentially hazardous situation that, if not avoided, could cause damage to the lift, vehicle and/or moderate injury.

CAUTION: Indicates a potentially hazardous electrical situation that, if not avoided, could cause damage to the lift, vehicle and/or moderate injury.

NOTE: Directions that make it easier to install or operate the lift.

NOTICE TO: Customers in the State of California

WARNING: Electrical cords, cables, product cords, wire assemblies, and other parts made with PVC can expose you to chemicals including DEHP, which are known to the State of California to cause cancer and birth defects or other reproductive harm. WASH HANDS AFTER HANDLING. For more information, go to www.P65Warnings.ca.gov.

1.0 PRODUCT REGISTRATION FORM

Please register your Bruno lift in one of two ways:

- Fill it out online at www.bruno.com/forms/product-registration
- Complete the enclosed Product Registration card and return via mail

Serial Number:

Please note a serial number is found on your Bruno lift. The serial and model number will be needed for your dealer to file a warranty claim, request parts or service the unit.

Authorized Bruno Dealer:

Bruno products must be installed and maintained by an authorized Bruno dealer, or the warranty will be null and void.

TABLE OF CONTENTS

1.0 Product registration form	3
2.0 For your safety	4
3.0 Loading a mobility device	5-6
Barrier belt (optional) operation	7-8
4.0 Unloading a mobility device.	9-11
5.0 Pinch points	12
6.0 Emergency backup operation.	13-14
7.0 Maintenance and troubleshooting	15
8.0 Bruno gold three-year limited warranty for Bruno vehicle lift	16

2.0 FOR YOUR SAFETY

- The Joey is designed to lift mobility devices weighing up to 350 lbs (159 kg).
- Do not overload lift or abuse the Joey.
- Do not remove any guards or covers from the Joey.
- Always be aware of surface conditions before using the Joey and take appropriate care.
- Park the vehicle on a horizontal, level surface. Load and unload on as level a surface as possible.
- Avoid unloading into vehicular traffic.
- Read and understand the operating instructions, and all safety precautions prior to using the Joey. Operate the lift several times while the installer is available to answer questions.
- Vehicle engine can be running while loading personal mobility device into vehicle, with transmission set in park (parking brake must be engaged on models with manual transmission).
- Keep hands, fingers and feet away from moving components during lift operation.
- Keep feet at a safe distance from the platform as it is lowered.
- Be sure the power source of the device being lifted (scooter, power chair, wheelchair) is OFF before lifting.

NOTE: The VSL-4400 is designed to operate at full rated capacity on a relatively flat, level surface.


It is designed to operate in temperatures above -25°F (-32°C). If operating in extreme heat (over 120°F/49°C) or extreme cold (below -25°F/-32°C), starting the vehicle prior to operating the VSL-4400 is recommended.

The VSL-4400 meets or exceeds F/CMVSS 302.

3.0 LOADING A MOBILITY DEVICE

■ STEP 3.1



Press the OUT switch and hold until the lift travels out and down to the ground.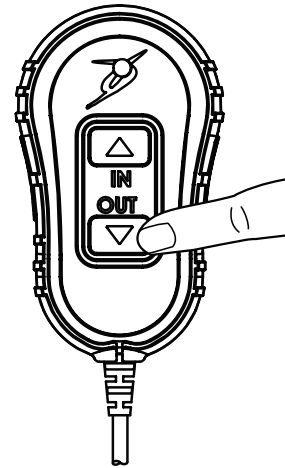

■ STEP 3.2


Release the switch when the platform rests flat on the ground.

NOTE: Continuing to press the switch after the platform is flat on the ground could cause the lift belt to unwind and reverse the direction of Up/Down.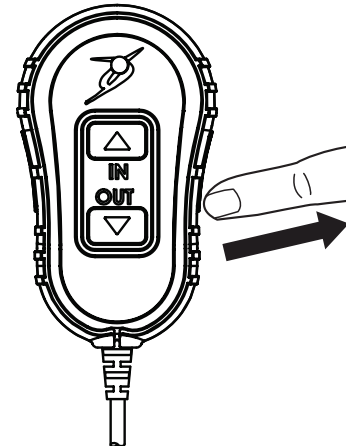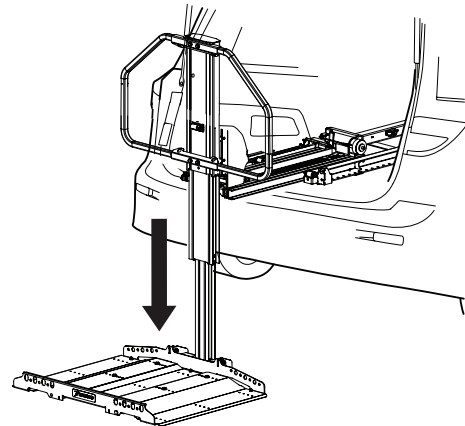

■ STEP 3.3

Position the mobility device on the platform.

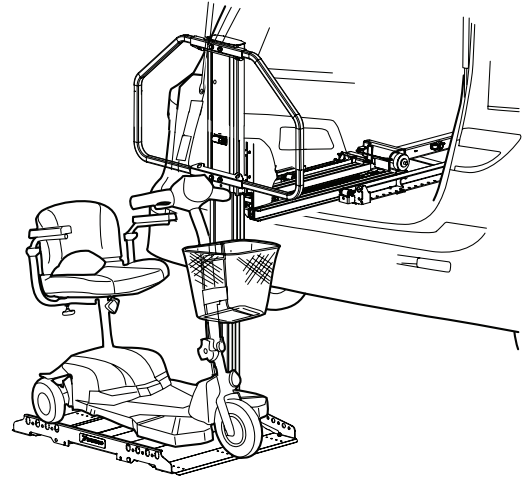


■ STEP 3.4

Press the IN switch until the lift reaches end of up travel. Release switch.

If equipped with optional barrier belts, place (2) belts across mobility device and hook onto platform. See next page for belt operation.

Press IN switch to power lift into the vehicle. The lift will stop automatically and the mast light will shut off.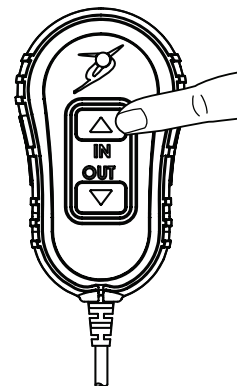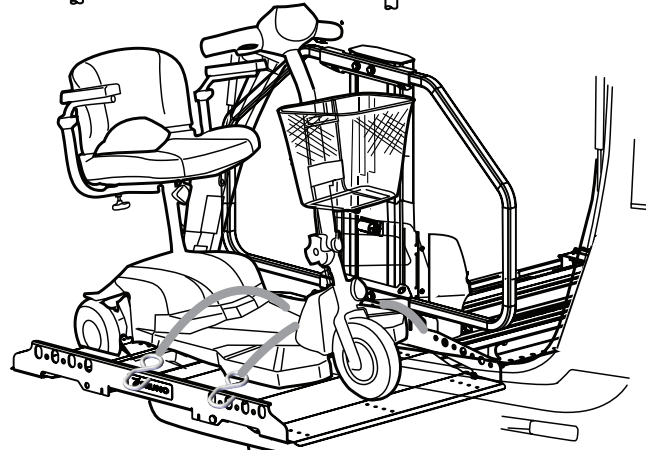
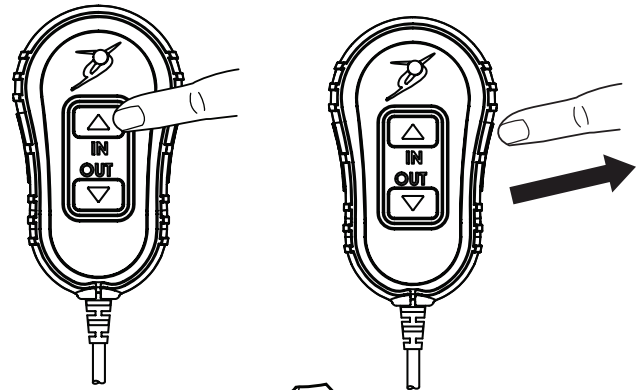


BARRIER BELT (OPTIONAL) OPERATION

■ STEP 3.4.1


- Press small tab (A).
- Push up large tab (B).
- Pull strap and hook to platform.
- Press and hold small tab (A) while pressing large tab (B) to home position.

NOTE: You will hear the tab click into its home position. The strap will “auto ratchet” to secure the mobility device.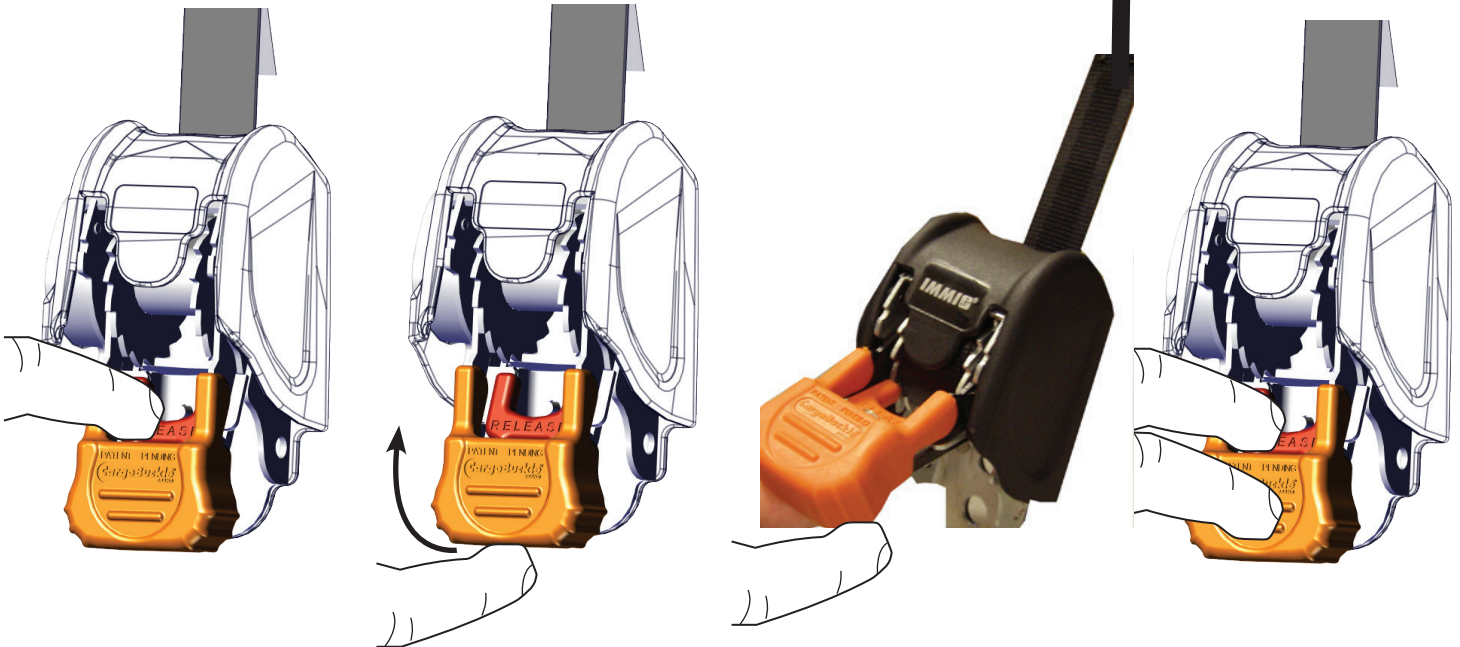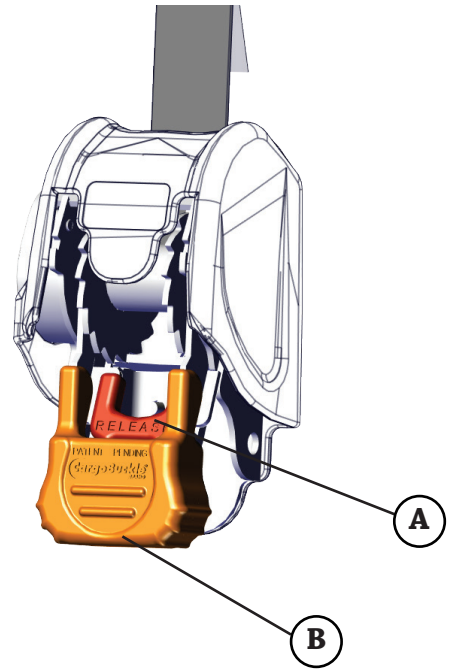


■ STEP 3.5

Store the remote on its mounting plate on the vehicle wall.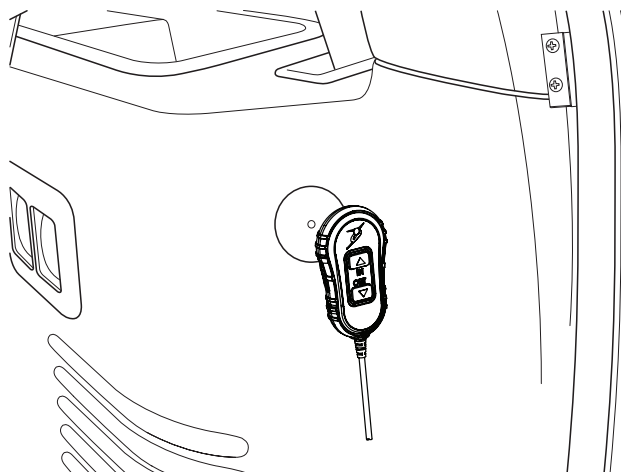

CAUTION: Make sure remote cord is clear from rear hatch to prevent cord from becoming pinched or severed.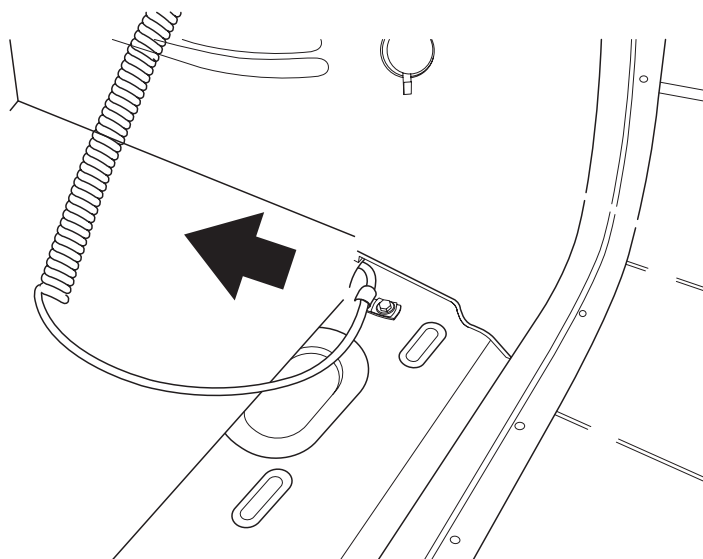


4.0 UNLOADING A MOBILITY DEVICE

■ STEP 4.1

Press and hold OUT switch until lift reaches end of outward travel. Release switch.



If equipped with optional barrier belts, unhook (2) belts securing mobility device to platform. See page 9 for belt operation.

Press and hold OUT switch to lower lift to ground. Release switch. Lift will stop automatically and mast light will shut off.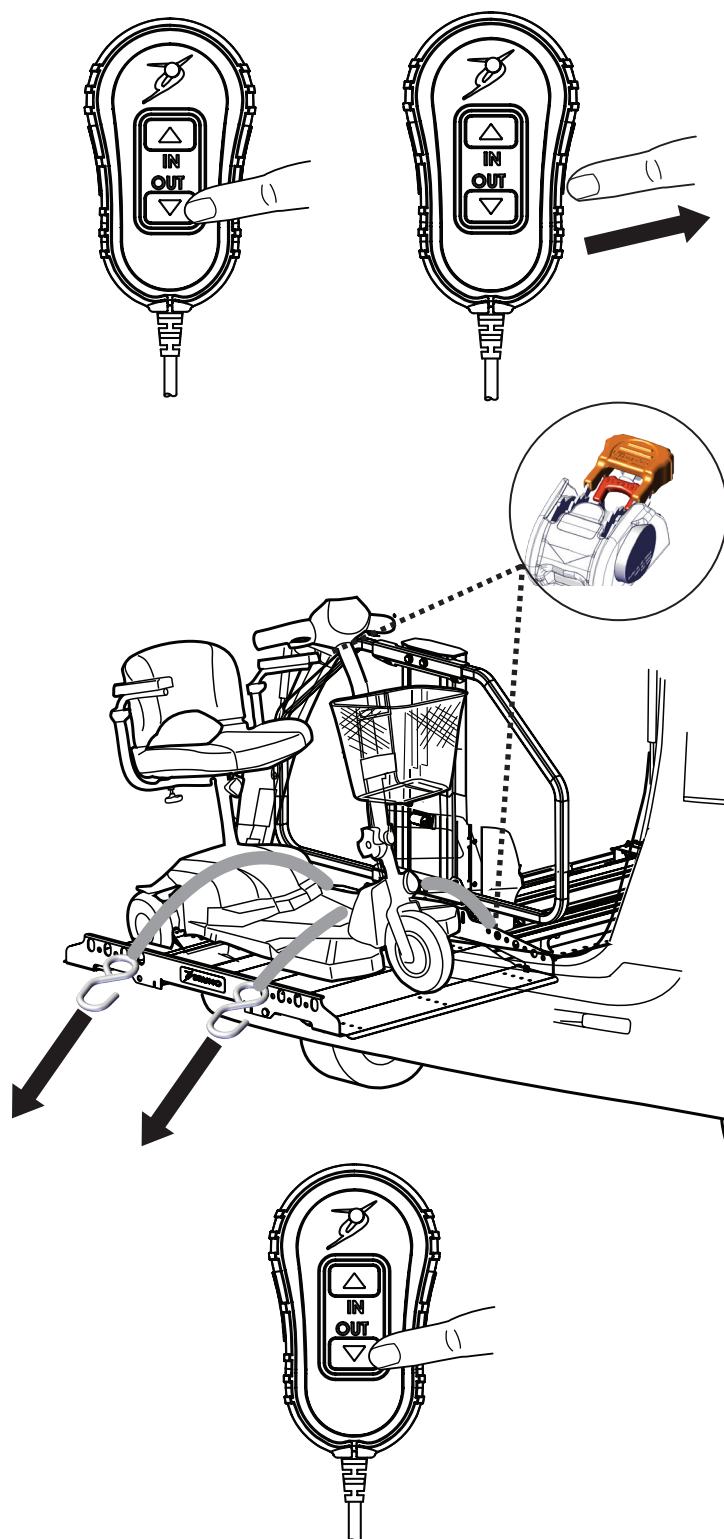


■ STEP 4.2

Release the switch when the platform rests flat on the ground.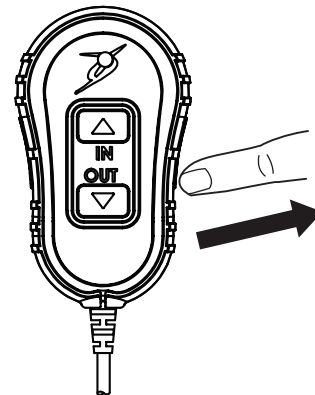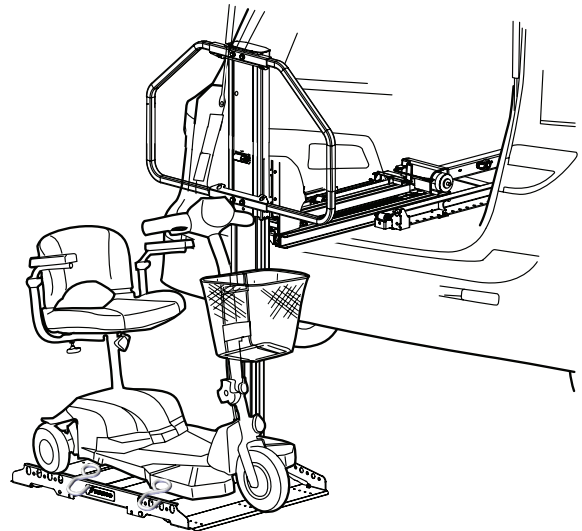

■ STEP 4.3


Remove the mobility device a safe distance from the platform.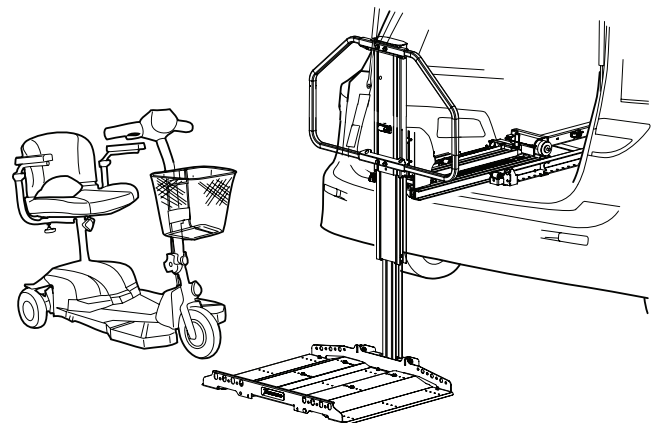

■ STEP 4.4

Press the IN switch until the lift travels up and into the vehicle. The lift will stop automatically and the mast light will shut off.


NOTE: The vehicle bumper may raise and/or lower slightly during lift operation with the mobility device on board. Do not be alarmed. This is normal.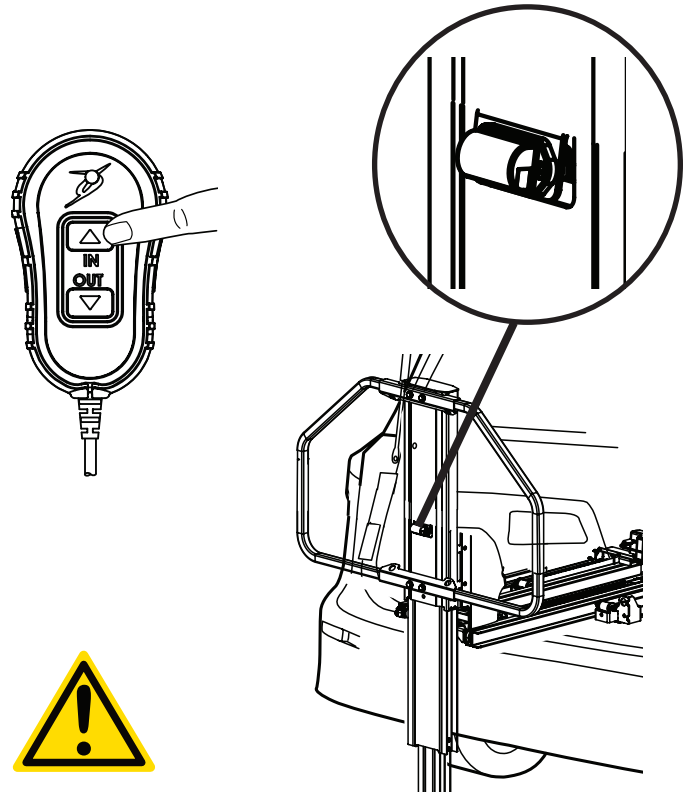

CAUTION: Make sure remote cord is clear from rear hatch to prevent cord from becoming pinched or severed.

■ STEP 4.5

Store the remote on its mounting plate on the vehicle wall.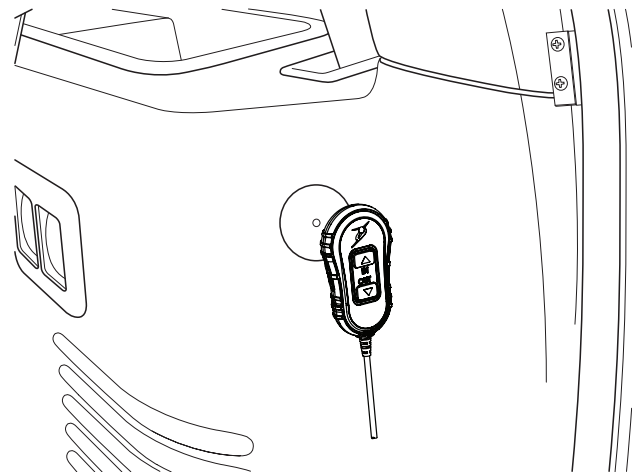




5.0 PINCH POINTS

Keep clear of pinch points.

Up/Down: Where the stages come together as the platform is lifted (A).

Between the barrier and platform (B)

In/Out: Where the inner stage meets up with the rear plate as the unit is retracted to home position (C).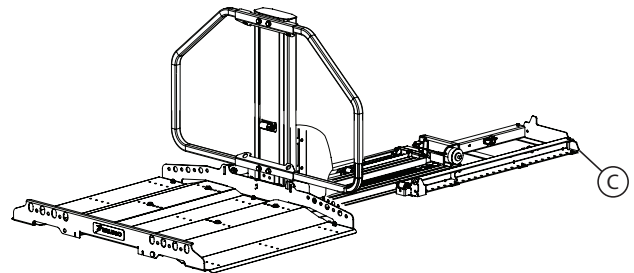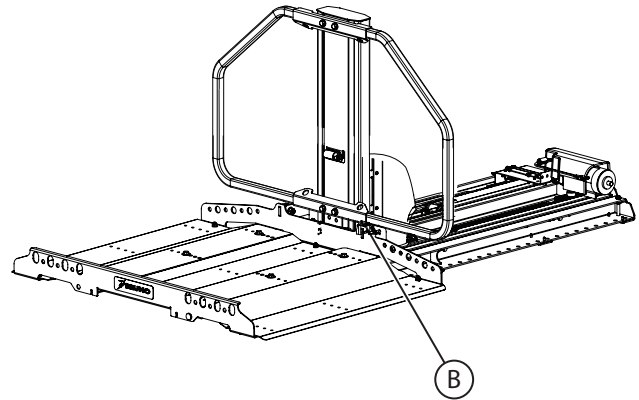
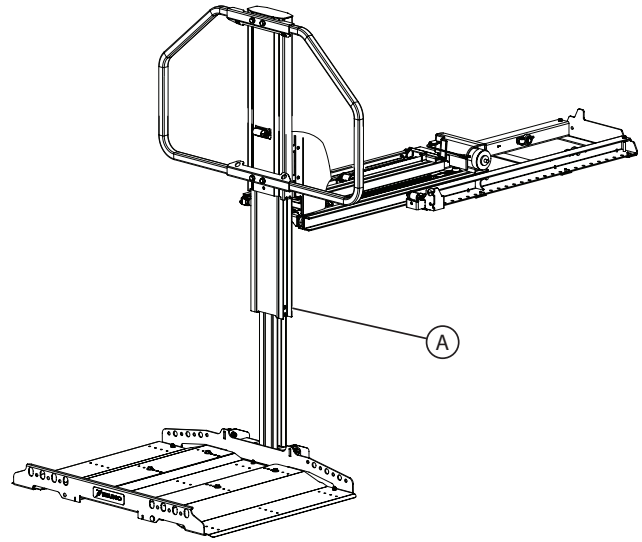


6.0 EMERGENCY BACKUP OPERATION

■ STEP 6.1


Pull in-line fuse (under hood) before proceeding with emergency back-up.

Locate back-up tool from bag containing operator's manual.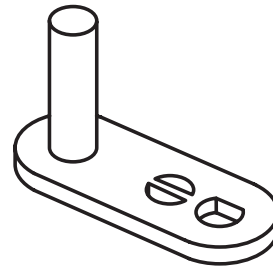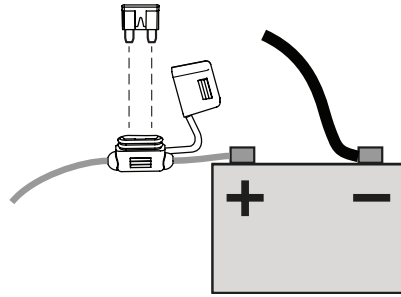


■ STEP 6.2

Remove black plastic caps from motor shafts.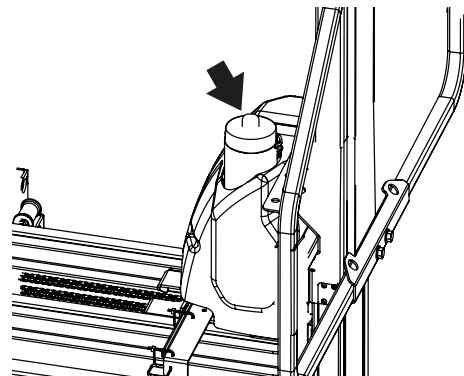

■ STEP 6.3

For scooters, secure one chock in front of front wheels and one chock behind rear wheels.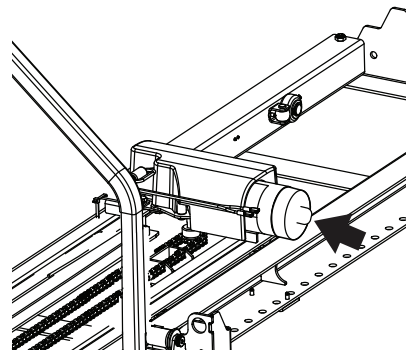




■ STEP 6.4

Place crank on motor #1. Turn crank clockwise to move platform all the way up.

If equipped with optional barrier belts, place (2) belts across mobility device and hook onto platform. See page 9 for belt operation.

Move crank to motor #2.

Turn crank clockwise to move platform all the way in.

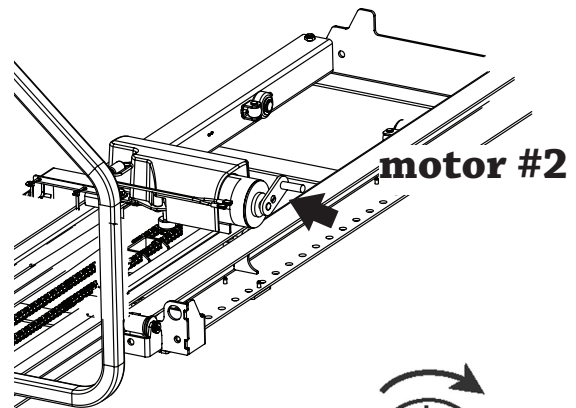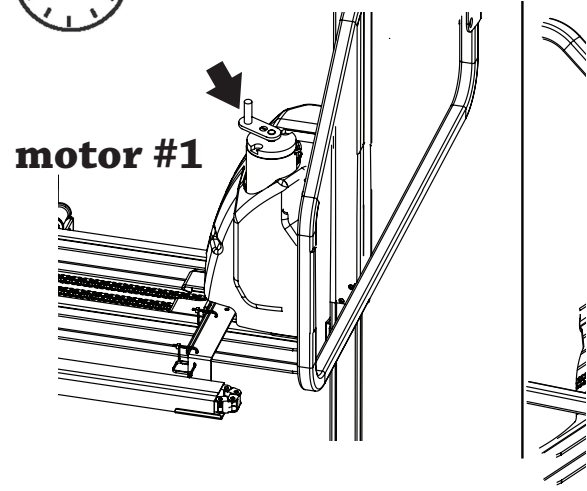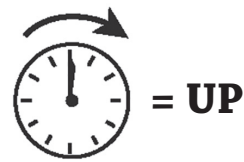■ STEP 6.5

Once stowed, have the lift inspected and repaired by a qualified Bruno dealer before further use.

7.0 MAINTENANCE AND TROUBLESHOOTING

Take the vehicle and lift to an authorized Bruno dealer for complete inspection and any necessary repairs/replacements once a year (more frequently if the lift is used under strenuous conditions).

PROBLEM	POSSIBLE CAUSE / SOLUTION
Lift does not operate.	+/- power connection to unit is loose. Verify that remote backlight is on. If unit is not in stowed position, verify that mast light is on.
Lift does not operate (will not run with rear vehicle door open)	Verify that the remote backlight is on.
	Dead vehicle battery. Charge, test and replace if necessary.
Lift operates slowly, lacks power or stalls	Weak vehicle battery. Attempt to start vehicle to determine if battery is weak. Charge battery before attempting to operate lift.
	Load is too heavy. Do not exceed maximum rated lift capacity of 350 lbs (159 kg).
	Vehicle parked on uneven ground. Move vehicle to level ground.
	Temperature in vehicle too hot or too cold. Start vehicle engine to lower/raise temperature.
Platform is not moving in direction indicated on remote.	Limit switch accidentally pressed. Reset circuit board by passing back of remote over door safety switch. Remote light should turn off, then back on.
	Top remote button should move platform UP and IN.
Platform has stopped moving but is not at top or bottom	Have a Bruno dealer inspect the lift and repair if necessary.
Lift travels out but not down	Check the OUT limit switch. Have unit adjusted and repaired if necessary by a Bruno dealer.
Lift stops at end of travel	To reset the printed circuit board (PCB), wave the remote by the door safety switch. The magnet in the remote will reset the PCB.

8.0 BRUNO GOLD LIMITED WARRANTY FOR VEHICLE LIFT

3 YEARS

Bruno Independent Living Aids, Inc.® (“Bruno”) is so confident in our USA designed and built units that we provide every original buyer with the following “Industry Leading” warranty for your peace of mind.

The quality of parts we use and the care our employees take allow Bruno to provide the original buyer of its Bruno Vehicle Lift with a **3-year warranty** that all parts are free of defects in material and workmanship.

Bruno’s warranty periods begin with the **date of the installation** of the Bruno Vehicle Lift.

Bruno Authorized Dealers are trained to evaluate your unique situation and then install the best Bruno product to meet your needs. The trained installation and service technicians employed by Bruno Authorized Dealers have the appropriate equipment and resources needed to correctly install and maintain your new vehicle lift and are the first contact for any warranty or service needs.

SPECIFICS OF THE WARRANTY COVERAGE:

- All parts are free of defects in material and workmanship for a period of three years after the date of installation.
 - All labor to address any product defect by a Bruno Authorized Dealer is covered for the first 30 days after installation.
-

WHAT IS NOT WARRANTY COVERAGE:

- Damage to units caused by misuse, abuse, accidents, neglect or modifications NOT made by Bruno Authorized Dealer.
 - Consumable items.
 - All labor by a Bruno Authorized Dealer after the first 30 days since installation.
 - Reasonable wear and tear.
-

ADDITIONAL NOTES:

All Service and Warranty issues are handled through the Bruno Authorized Dealer that installed the unit.

- In no event shall Bruno be responsible for indirect, incidental or consequential damages, whether such damages arise based on claims based on contract, warranty, tort (including negligence, strict liability or product liability).
- Bruno reserves the right to replace, repair or, if not commercially practical and/or feasible, to elect to refund the purchase price at its sole discretion.
- This warranty gives the original buyer specific legal rights, and you may also have other rights which vary from state to state. Bruno specifically does not authorize any person to extend the time or scope of this warranty.

For more detailed information regarding this limited warranty, please contact Bruno by writing:

Bruno Independent Living Aids, Inc. | Attn: Service Department | 1780 Executive Drive |
Oconomowoc, WI 53066 USA

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Bruno Independent Living Aids, Inc.MD